Z-CAD[®] Deep liquid

EN	ENGLISH	Operating instructions:	2
DE	DEUTSCH	Gebrauchsanweisung:	5
FR	FRANÇAIS	Mode d'emploi:	8
IT	ITALIANO	Istruzioni per l'uso:	12
ES	ESPAÑOL	Instrucciones de uso:	16
PT	PORTUGUÊS	Manual de instruções:	20

CE 0297

EC REP **EC IMP**

METOXIT AG
Emdwiesenstrasse 6
CH-8240 Thayngen
Switzerland
Info@metoxit.com
www.metoxit.com
SRN CH-MF-000019683

M+S Ceramics Kft.
Rozmaring utca 13
HU-8000 Székesfehérvár
Hungary
Info@m-s-ceramics.com
SRN HU-AR-000006710 (EC-REP)
SRN HU-IM-000006711 (EC-IMP)

EN ENGLISH

Operating instructions

01. Product description

Z-CAD® Deep liquid products are ready to use and are suitable for use with Metoxit Z-CAD® CAM blanks without pre-colouring.

02. Product designation

Z-CAD® Deep liquid

03. Indication for use for colouring with Metoxit Z-CAD® Deep liquid

Restorations milled from Metoxit® Z-CAD® CAM blanks can be coloured with Metoxit Z-CAD® liquids. Z-CAD® Deep liquid is specifically designed for the dipping method.

04. Correct usage

Z-CAD® Deep liquid dipping technique for individualisation with ceramic layer.

Clean the milled zirconium oxide units well.

Remove dust with a dry brush and blow it off with compressed air (low pressure and oil-free).

Clean scrupulously before dipping.

Dip into the Z-CAD® Deep liquid with plastic tweezers.

Never use metal tweezers.

Protect the container with the liquid against opening.

Transfer the necessary quantity into a beaker.

The edge of the framework shows best facing upwards.

Leave the framework in the Z-CAD® Deep liquid for 5-10 minutes.

A longer dipping time will not negatively affect the colouring result.

Dry for 12 minutes on a paper towel under an infra-red heat lamp.

Then place the framework on the sinter balls with the edge facing upwards.

The sinter program is taken from the operating instructions for Metoxit Z-CAD® CAM blanks.

It is recommended to use this program exactly.

Before further processing, please clean thoroughly with water.

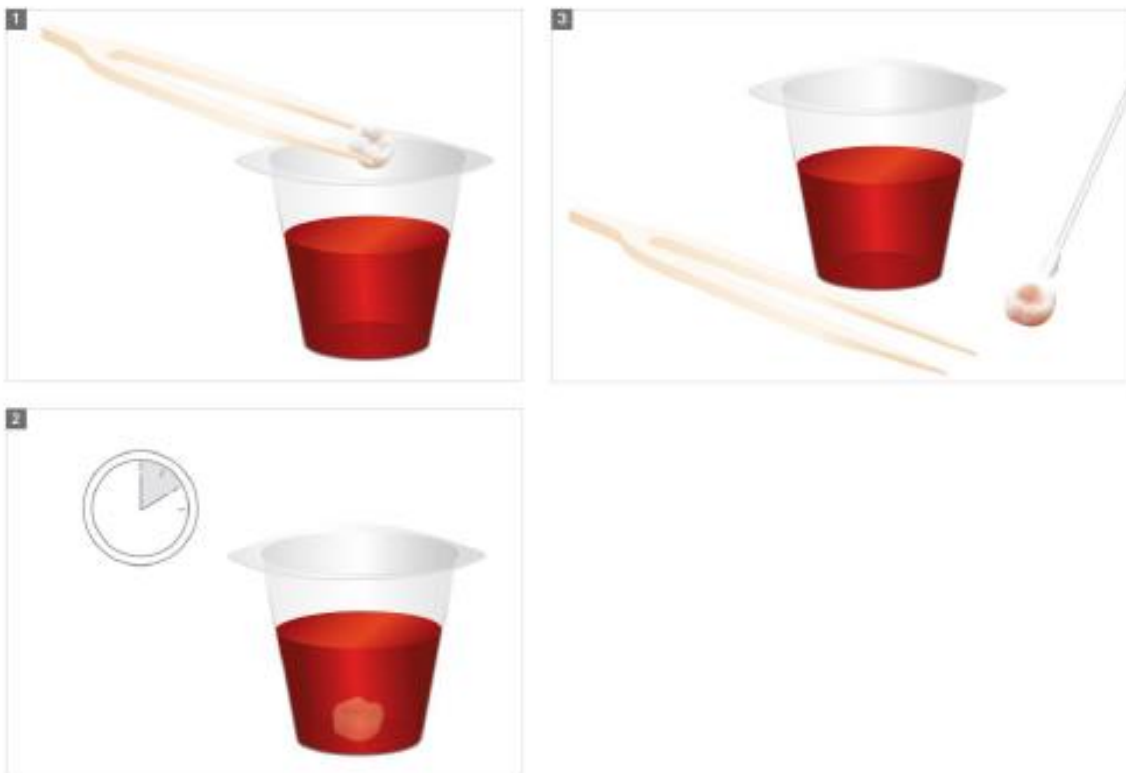
Illustration: Dipping method

05. Indications

The Z-CAD® Deep liquid is used to colour pre-sintered crowns, inlays, onlays, veneers and bridges for dental restorations from Metoxit® Z-CAD® CAM blanks.

06. Target patient group and intended users

Patients with a tooth defect or missing teeth, with erupted permanent dentition and adequate dental care, good health and no contraindications. The intended users are dentists/dental technicians who are thoroughly trained in dental restoration. Dental specialists must observe the local provisions and restrictions applicable to their profession.

Metoxit® Z-CAD® Deep liquid is not intended for general use or as a commercially available product.

07. Contraindications

Z-CAD® Deep liquid should not be used for colouring after dipping.

08. Warnings and precautionary measures

Hazard Statement(s): H319: Causes serious eye irritation.

Precautionary statement(s): P305 + P351 + P338:

In case of contact with the eyes:

Rinse cautiously with water for several minutes. If possible, remove existing contact lenses and continue rinsing.

Signal word: Warning

Hazard pictogram: see product label.

GHS08

Observe the information in the current safety data sheet in the download section of Metoxit AG, www.metoxit.com/unternehmen/download

09. Chemical composition

weight %					
Aluminium nitrate	< 0.3	Iron nitrate	< 0.6	Citric acid	< 0.02
Ammonium iron citrate	< 1	Erbium nitrate	< 2	Food colouring	< 0.5
Cobalt nitrate	< 0.05	Manganese nitrate	< 0.005	Water	< 100

10. Caution

Z-CAD® Deep liquid may only be sold to or on the order of dentists/dental technicians and Z-CAD® Deep liquid should not be used after the expiry date. See packaging labelling.

11. Storage and handling

Before using for the first time, please check that the packaging and contents are intact. Check that the statement on the label and the Z-CAD® Deep liquid are the same. Protect the product from sunlight and heat, avoid damage and contact with other liquids. Z-CAD® Deep liquid should be closed immediately after use and can be used again until the expiry date of the product. In the event of shock to the dental restorations, some cracks may occur and some thin parts in the zirconium restoration may break. It is therefore recommended to treat the restoration with care to prevent severe impacts. Carry out a visual check and, in case of doubt after a shock or impact, do not process further (risk of cracks when sintering).

12. Note to users and patients

Serious incidents related to the product must be reported to the manufacturer or its EC representative and to the competent authority of the Member State in which the user or patient is located.

13. Magnetic Resonance Imaging (MRI)

Z-CAD® Deep liquid products are manufactured from a material that is not affected by MRI and is MR safe. However, restorations treated with this material can be fitted with metal implants, abutments and screws that can be affected by MRI energy.

14. Further information

Additional information on Metoxit AG products can be found on the Metoxit website www.metoxit.com where relevant. Operating instructions can be downloaded from www.metoxit.com/unternehmen/download. Please note that dental restorations produced from Z-CAD® CAM blanks and treated with Z-CAD® Deep liquid are considered custom-made products.

15. Disposal

Disposal should be done in an environmentally friendly way in accordance with local regulations.

16. Information that must be provided to patients

Information on contraindications, warnings, preventative measures, side effects and complications with Metoxit products should be provided to patients.

Please note that the dental restoration is milled and where necessary, treated with Z-CAD® Deep liquid. The patient must be informed on the MRI compatibility of the Metoxit products used.

17. Validity

On the publication of these operating instructions, all previous versions lose their validity.

18. Availability

Some Metoxit AG products are not available in all countries.

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20. Trademark

Z-CAD® is a registered name and trademark of Metoxit AG, Switzerland

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Alguns produtos Metoxit AG não estão disponíveis em todos os países.
Z-CAD® é uma marca registada e marca comercial da Metoxit AG.