# Z-CAD® Surface liquid

|           |           |                             |    |
|-----------|-----------|-----------------------------|----|
| <b>EN</b> | ENGLISH   | Operating instructions:     | 2  |
| <b>DE</b> | DEUTSCH   | Gebrauchsanweisung: .....   | 8  |
| <b>FR</b> | FRANÇAIS  | Mode d'emploi: .....        | 14 |
| <b>IT</b> | ITALIANO  | Istruzioni per l'uso: ..... | 21 |
| <b>ES</b> | ESPAÑOL   | Instrucciones de uso: ..... | 28 |
| <b>PT</b> | PORTUGUÊS | Manual de instruções: ..... | 35 |

**CE** 0297

METOXIT AG  
Emdwiesenstrasse 6  
CH-8240 Thayngen  
Switzerland  
Info@metoxit.com  
www.metoxit.com  
SRN CH-MF-000019683

**EC REP** **EC IMP**

M+S Ceramics Kft.  
Rozmaring utca 13  
HU-8000 Székesfehérvár  
Hungary  
Info@m-s-ceramics.com  
SRN HU-AR-000006710 (EC-REP)  
SRN HU-IM-000006711 (EC-IMP)

# EN ENGLISH

## Operating instructions

### 01. Product description

Z-CAD® Surface liquid products are ready to use and are suitable for use with Metoxit Z-CAD® CAM blanks without pre-colouring.

### 02. Product designation

### Z-CAD® Surface liquid

### 03. Indication for use for colouring with Metoxit Z-CAD® Surface liquid

Restorations milled from Metoxit® Z-CAD® CAM blanks can be coloured with Metoxit Z-CAD® liquids.

Z-CAD® Surface liquid is primarily used for individual colouring and characterisation with the brush technique

### 04. Correct usage

Z-CAD® Surface liquid is used in the brush technique for monolithic work, and is primarily used for Metoxit Z-CAD® One4All and Z-CAD® Smile.

Clean the milled zirconium oxide units well.

Remove dust with a dry brush and blow it off with compressed air (low pressure and oil-free).

The milled zirconium oxide units must be completely dry before painting.

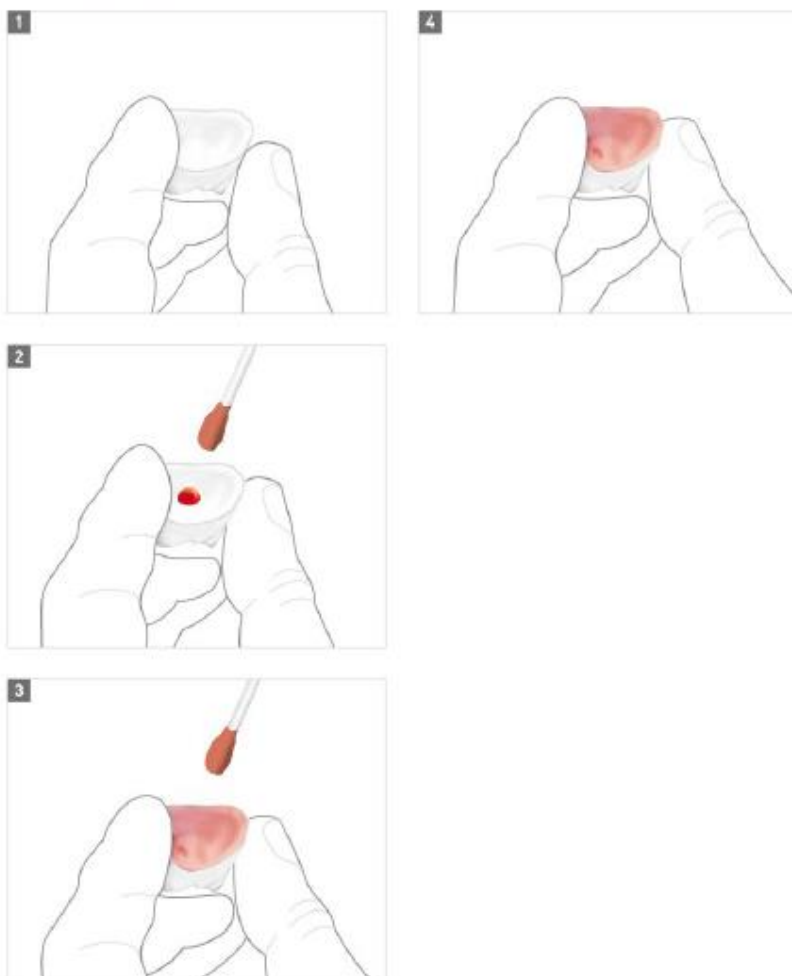
### Process step 1

Apply Z-CAD® Surface liquid (dentine colour with red pigment) from the inside with a brush or cotton bud.

To do this, place a few drops in the inside and brush up to the edge of the crown.

Repeat this process until the red colour in the incisal area is visible from the outside.

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## Process step 2

Colour the individual characteristics depending on the patient's situation of the cervical part, neck of the tooth and the fissures according to the dentine colour, or accentuate with Special Liquid.

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
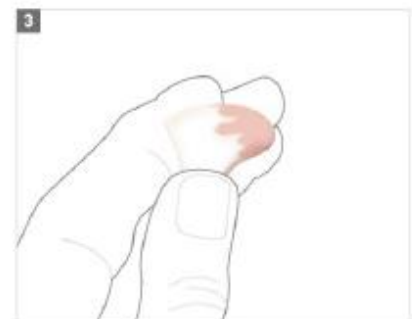
Illustration: Process step 2, brush technique

### Process step 3

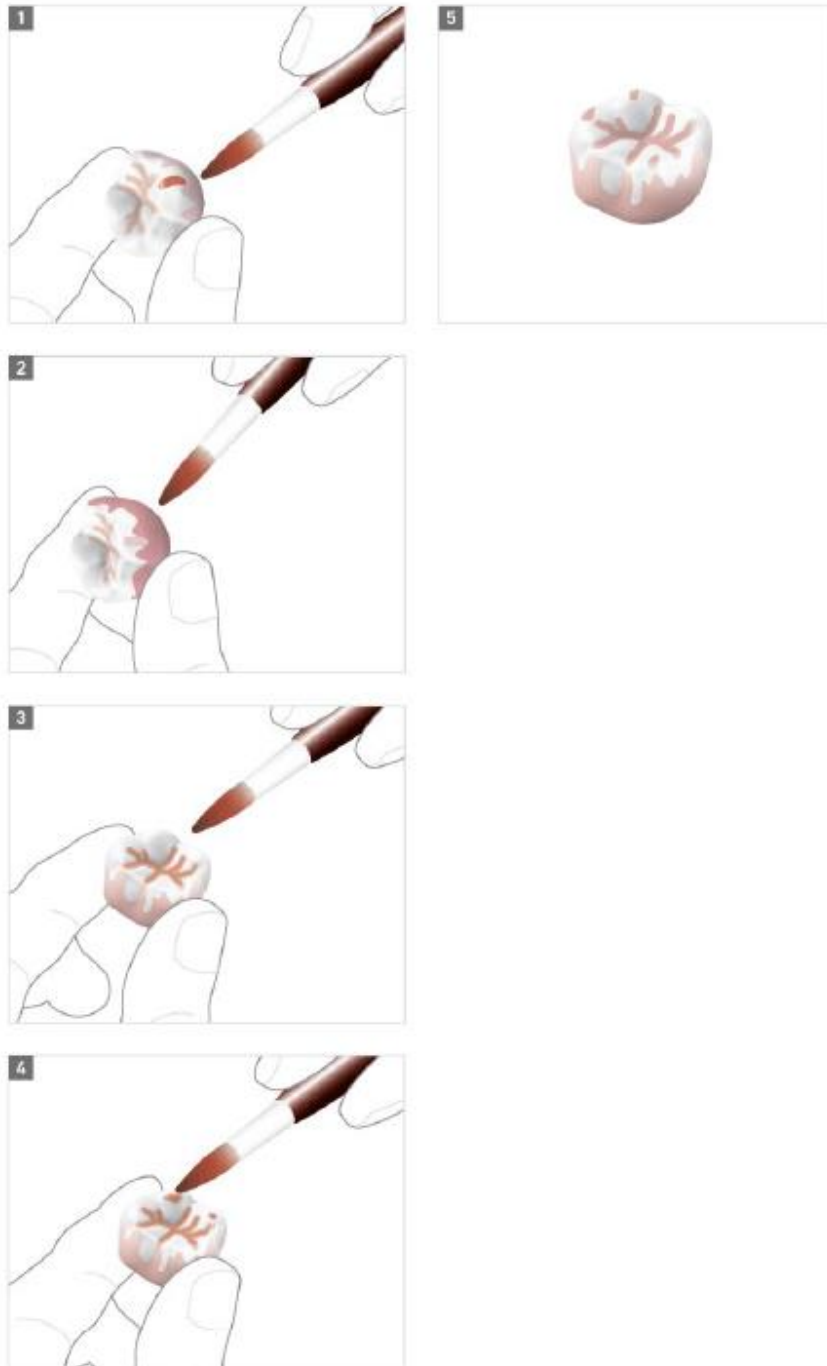Colour in the base colour of the dentine area and individualise mamelons, proximal surfaces or eroded cusps. If desired, apply some brush strokes on the outside of the crown from the edge. The incisal areas can be further individualised from the outside.

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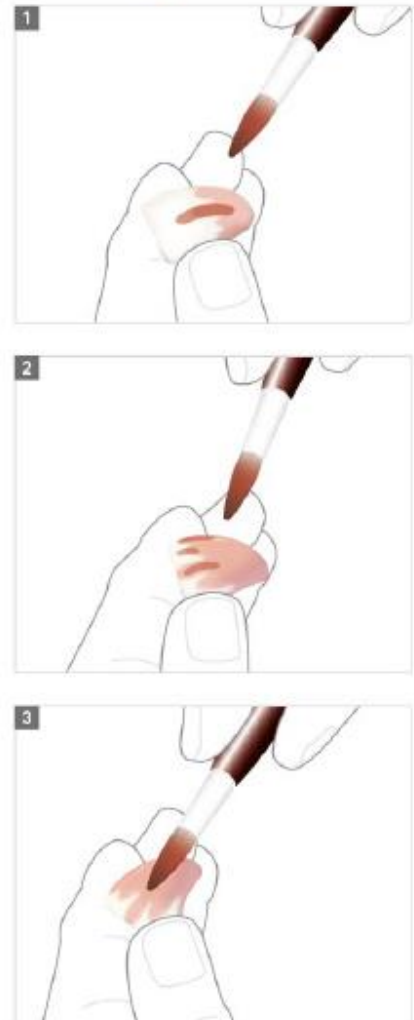
Illustration: Process step 3, brush technique

## Process step 4

Individualise the incisal/enamel bar area to match the remaining dentition with the respective blue, grey Special Liquid.

**TIP:** for anterior teeth, labially paint the "enamel bar" alternately in grey and blue.

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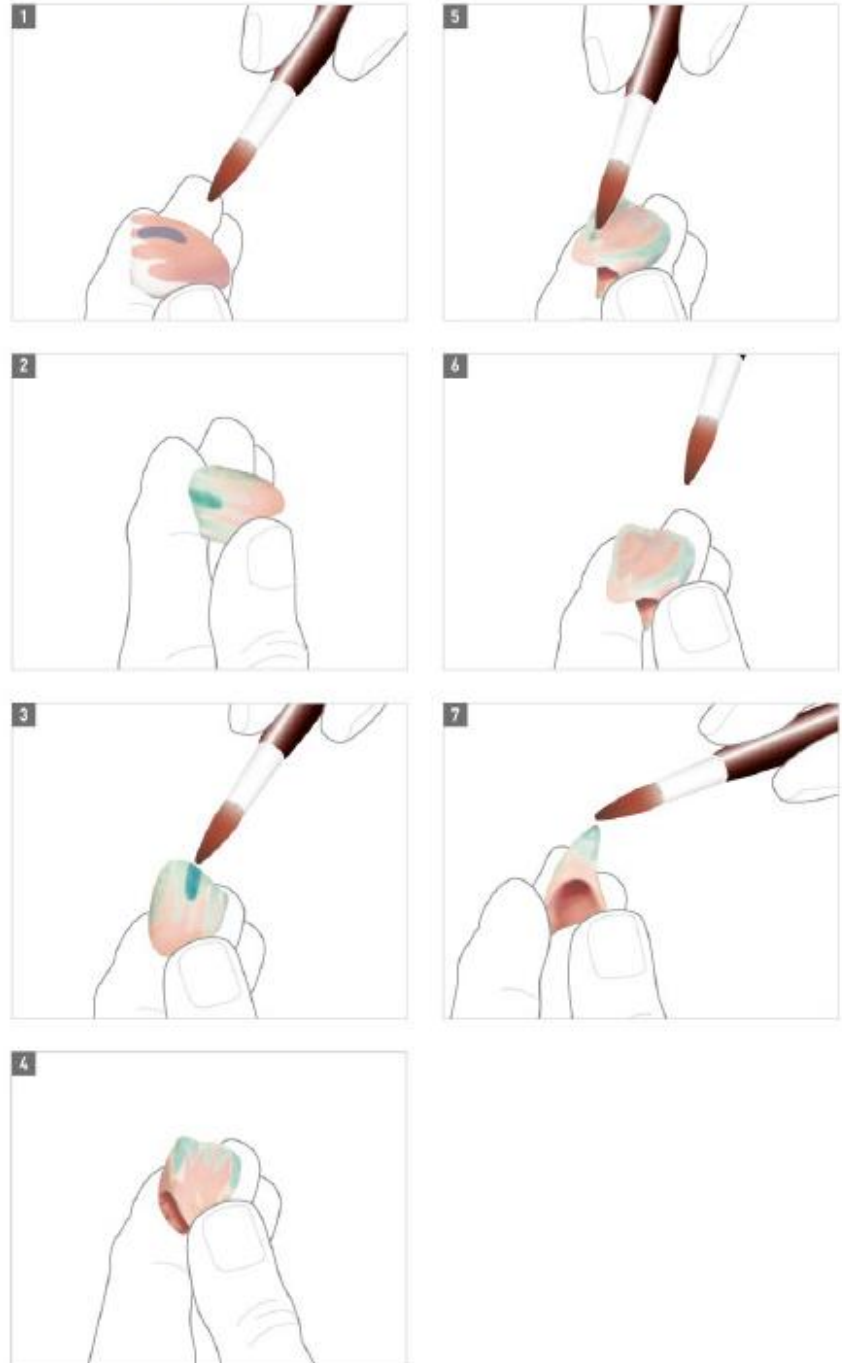
Illustration: Process step 4, brush technique

## 05. Proper use after treatment with Z-CAD® Surface liquid

Dry for 12 minutes on a paper towel under an infra-red heat lamp.  
Then place the framework on the sinter balls with the edge facing upwards.  
The sinter program is taken from the operating instructions for Metoxit Z-CAD® CAM blanks.  
It is recommended to use this Metoxit program exactly.  
Before further processing, please clean thoroughly with water.

## 06. Indications

The Z-CAD® Surface liquid is used to colour pre-sintered crowns, inlays, onlays, veneers and bridges for dental restorations from Metoxit® Z-CAD® CAM blanks.  
Z-CAD® Surface liquid is primarily used for processing Metoxit Z-CAD® One4All and Z-CAD® Smile.

## 07. Target patient group and intended users

Patients with a tooth defect or missing teeth, with erupted permanent dentition and adequate dental care, good health and no contraindications. The intended users are dentists/dental technicians who are thoroughly trained in dental restoration. Dental specialists must observe the local provisions and restrictions applicable to their profession.  
Metoxit® Z-CAD® Surface liquid is not intended for general use or as a commercially available product.

## 08. Contraindications

Z-CAD® Surface liquid should not be used for colouring after dipping.

## 09. Warnings and precautionary measures

Hazard Statement(s): H319: Causes serious eye irritation.  
Precautionary statement(s): P305 + P351 + P338:  
In case of contact with the eyes:  
Rinse cautiously with water for several minutes. If possible, remove existing contact lenses and continue rinsing.  
Signal word: Warning  
Hazard pictogram: see product label.

GHS08

Observe the information in the current safety data sheet in the download section of Metoxit AG, [www.metoxit.com/unternehmen/download](http://www.metoxit.com/unternehmen/download)

## 10. Chemical composition

| weight %              |        |                   |         |                |        |
|-----------------------|--------|-------------------|---------|----------------|--------|
| Aluminium nitrate     | < 0.3  | Iron nitrate      | < 0.6   | Citric acid    | < 0.02 |
| Ammonium iron citrate | < 1    | Erbium nitrate    | < 2     | Food colouring | < 0.5  |
| Cobalt nitrate        | < 0.05 | Manganese nitrate | < 0.005 | Water          | < 100  |

## 11. Caution

Z-CAD® Surface liquid may only be sold to or on the order of dentists/dental technicians and Z-CAD® Surface liquid should not be used after the expiry date. See packaging labelling.

## 12. Storage and handling

Before using for the first time, please check that the packaging and contents are intact. Check that the statement on the label and the Z-CAD® Surface liquid are the same. Protect the product from sunlight and heat, avoid damage and contact with other liquids. Z-CAD® Surface liquid should be closed immediately after use and can be used again until the expiry date of the product. In the event of shock to the dental restorations, some cracks may occur and some thin parts in the zirconium restoration may break. It is therefore recommended to treat the restoration with care to prevent severe impacts. Carry out a visual check and, in case of doubt after a shock or impact, do not process further (risk of cracks when sintering).

## 13. Note to users and patients

Serious incidents related to the product must be reported to the manufacturer or its EC representative and to the competent authority of the Member State in which the user or patient is located.

#### 14. Magnetic Resonance Imaging (MRI)

Z-CAD® Surface liquid products are manufactured from a material that is not affected by MRI and is MR safe. However, restorations treated with this material can be fitted with metal implants, abutments and screws that can be affected by MRI energy.

#### 15. Further information

Additional information on Metoxit AG products can be found on the Metoxit website [www.metoxit.com](http://www.metoxit.com) where relevant. Operating instructions can be downloaded from [www.metoxit.com/unternehmen/download](http://www.metoxit.com/unternehmen/download). Please note that dental restorations produced from Z-CAD® CAM blanks and treated with Z-CAD® Surface liquid are considered custom-made products.

#### 16. Disposal

Disposal should be done in an environmentally friendly way in accordance with local regulations.

#### 17. Information that must be provided to patients

Information on contraindications, warnings, preventative measures, side effects and complications with Metoxit products should be provided to patients. Please note that the dental restoration is milled and where necessary, treated with Z-CAD® Surface liquid. The patient must be informed on the MRI compatibility of the Metoxit products used.

#### 18. Validity

On the publication of these operating instructions, all previous versions lose their validity.

#### 19. Availability

Some Metoxit AG products are not available in all countries.

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#### 21. Trademark

Z-CAD® is a registered name and trademark of Metoxit AG, Switzerland

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Alguns produtos Metoxit AG não estão disponíveis em todos os países.
**Z-CAD®** é uma marca registada e marca comercial da Metoxit AG.